



Hunter Medical, LLC
1116 W. 7th St. PMB 35
Columbia, TN 38401 USA

Phone 877-721-9988
www.huntermed.net

Hunter Medical ElbowLOC Surgical Hand Table

Carbon Fiber Arm Table with Leg Support

REF: 22187-2300



Rx Only



How to Use: Attach the table to the rail of the OR table or stretcher and tighten the clamps to prevent the table from moving or becoming detached from the table during a procedure. The table must be draped prior to any procedures.

Contact Hunter Medical for pricing and stock number information on all items below.

PD-100	2" Pad Contoured Foam Pad
SL-2001	Standard Single Post Leg
K-708	Under Pad Mounting Bracket

Cleaning Recommendations: Products above may be cleaned with any standard disinfectant. After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.



If the table is not properly assembled or is damaged, **DO NOT USE!** Contact Hunter Medical for assistance with any necessary repairs or replacement parts. Maximum weight capacity is 50lbs (22kg).