



Hunter Medical, LLC
1116 W. 7th St. PMB 35
Columbia, TN 38401 USA

Phone 877-721-9988
www.huntermed.net

Nylon Finger Trap

Finger Traction Device



Rx Only



REF: 22187-2401, 22817-2402, 22817-2403, 22817-2404

How to Use: Place patient's fingers in finger traps. Once the fingers are secure, place the connecting rope of the traps onto the hook of the Hunter Medical Wrist Tower Traction Gauge or other traction device.

Sterilization Instructions: Clean with any standard disinfectant prior to sterilization.



Sterilize the finger trap before each use.

1. Steam Sterilization General Instructions:

Follow standard hospital sterilization practices

- A. Place clean instruments in an instrument tray
- B. If instruments are to be wrapped, two double thickness of #140 thread count wrappers should be utilized. Exposure times are the same for wrapped or unwrapped instruments
- C. Follow the instructions for the gravity air displacement steam sterilization listed below

2. Gravity Steam Sterilization:

- A. Double wrap in standard central supply wrap.
- B. Set the temperature and corresponding exposure time:
121-123 degrees Celsius for 30 minutes

After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect finger trap before and after each use.



If the Finger Trap is split or frayed, **DO NOT USE**. Handle and dispose of in accordance with acceptance medical practice and applicable laws and regulations.